


PERI-IMPLANTITIS

Nonsurgical treatment

Implant-Safe Protocol

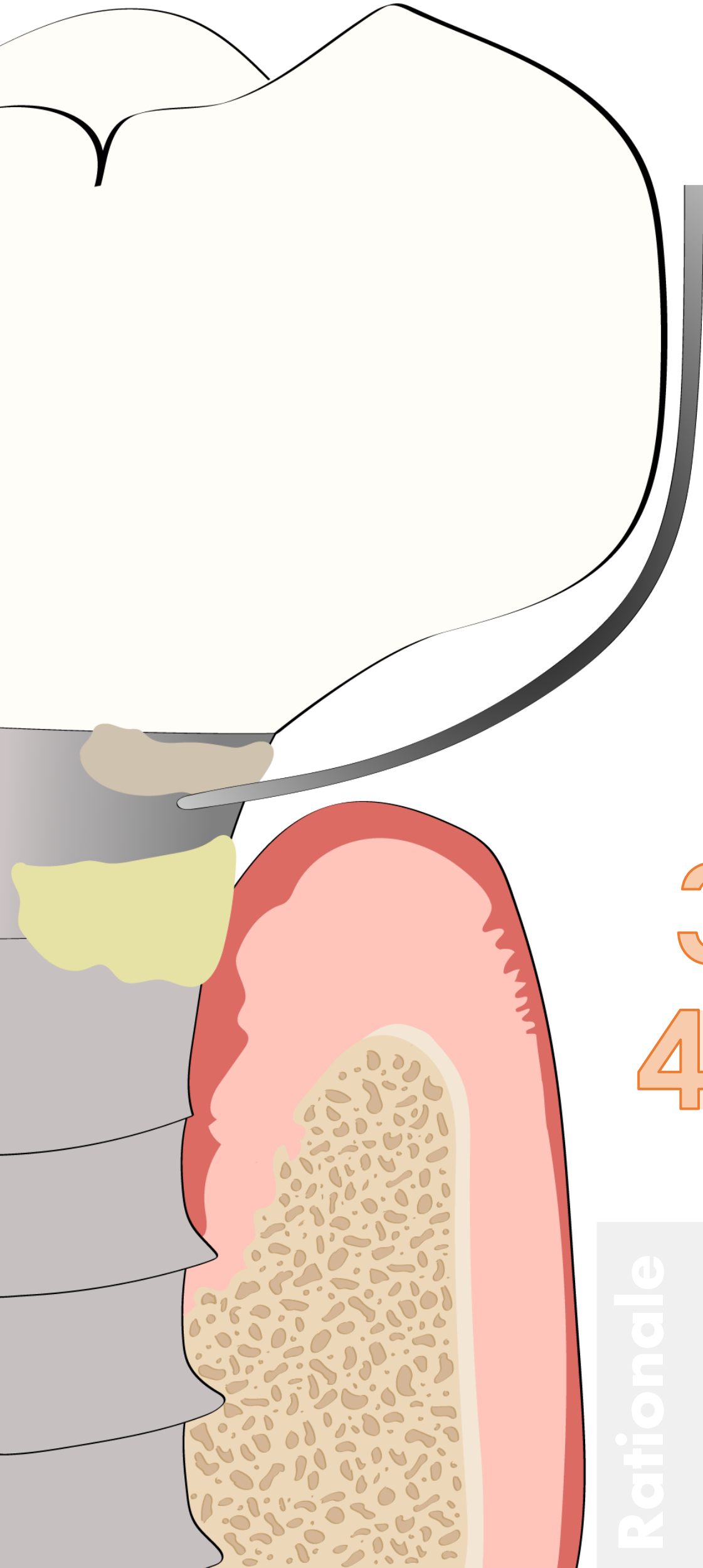

1 Probe the sulcus and assess for calculus

Impla-Safe Probe to avoid tissue seal rupture



No calculus

Calculus Present

Remove calculus with *Impla-Safe Soft Curette*
*only on abutments – never on implant threads



2 Detection of residual cement

Impla-Safe Cement Explorer



Not detected

Residual Cement Present


Chip off cement with *Impla-Safe Cement Remover*




3 Subgingival irrigation with saline

4 Shoe-shine implant surface

Impla-Safe Ti Micropliers to safely push gauze soaked in saline in the sulcus

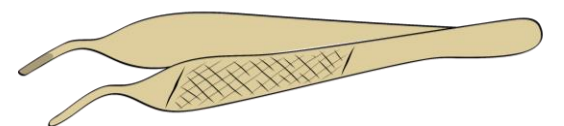

Rationale

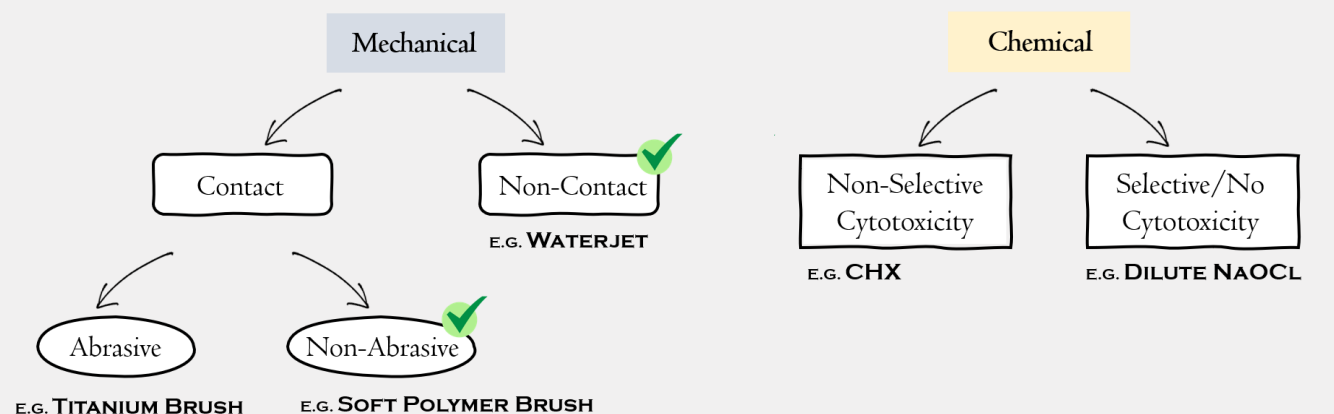

NEW!

Impla-Safe™

Peri-Implant Management Set

For Prevention And Non-Surgical Management Of Peri-Implantitis

Designed By Dr. George Kotsakis



**Impla-Safe™
Perio Probe**

Titanium
Flexible
Ultra Thin
#IMPLA-SAFE-PROBE
.....99.00 EACH



**Impla-Safe™
Mirror #4**

Double Sided
#IMPLA-SAFE-MIRROR
..... 99.00 BOX OF 6

**Impla-Safe™
Mirror Handle**

#MIRROR-HNDL-BLK
.....65.00 EACH



**Impla-Safe™
Soft Curette 3/4**

Titanium
For Supragingival Calculus
#IMPLA-SAFE-SOFT-TI-CU
.....99.00 EACH



**Impla-Safe™
Cement Explorer™**

Titanium
High Modulus Of Elasticity For
Optimal Tactile Sensitivity
#IMPLA-SAFE-EXPLORER
.....99.00 EACH



**Impla-Safe™
Cement Remover**

Titanium
Micro-Serrated For Efficient
Cement Removal
#IMPLA-SAFE-CEM-REMOVER
.....99.00 EACH



**Impla-Safe™
Titanium Micropliers -
Curved**

Titanium
#IMPLA-SAFE-MICROPLIERS
.....129.00 EACH

Everything For Your Implant Practice But The Implants®

Salvin Dental Specialties, Inc • Toll Free (US & Canada) 800-535-6566 • www.salvin.com

Phone 704-442-5400 • Fax 704-442-5424 • Email orders@salvin.com

© 2023 Salvin Dental Specialties, Inc. All Rights Reserved. REV. 11-2023

PERI-IMPLANTITIS NON-SURGICAL TREATMENT – IMPLA-SAFE PROTOCOL

1 Probe The Sulcus & Assess For Calculus Using Impla-Safe Probe To Avoid Tissue Seal Rupture



If Calculus Present



Remove Calculus Using Impla-Safe Soft Curette
(Only On Abutments – Never On Implant Threads)

2 Use Impla-Safe Cement Explorer For Detection Of Residual Cement



If Residual Cement Is Present



Use Impla-Safe Cement Remover
To Chip Off Cement

3 Subgingival Irrigation With Saline

4 Shoe-Shine Implant Surface – Use Impla-Safe Ti Micropliers To Safely Push Gauze Soaked In Saline In The Sulcus



"Impla-Safe is the first fully implant-specific instrument set for peri-implant maintenance, and for the prevention and non-surgical management of peri-implant inflammation. From the flexible probe to the soft titanium alloy instruments for implant-specific factors, such as excess cement, this set addresses the iatrogenic damage caused to implants by dental instruments and scaling using an evidence-based workflow. Should be integral in the maintenance of patients with dental implants."

Dr. George Kotsakis
Periodontist, Hoboken, NJ



"These exceptionally well crafted implant site specific instruments for detecting and removing excess cement are a brilliant innovation and must have for all implant surgically based clinicians."

Dr. Michael A. Pikos
Oral Surgeon, Palm Harbor, FL



"The Impla-Safe set is a game-changing, revolutionary implant-specific solution!"

Dr. Jennifer Hirsch Doobrow
Periodontist, Cullman, AL



"I'm excited to see an implant specific solution with instruments that adapt to the shape of implant-supported restorations and can help detect excess cement during and after cementation."


Dr. Konstantinos Chochlidakis
Prosthodontist, Rochester, NY



"The Impla-Safe Protocol and instrument set addresses what has been missing in peri-implantitis treatment: an evidence-based, implant-specific solution for our patients."

Dr. Eduardo R. Lorenzana
Prosthodontist, San Antonio, TX

SPECIAL

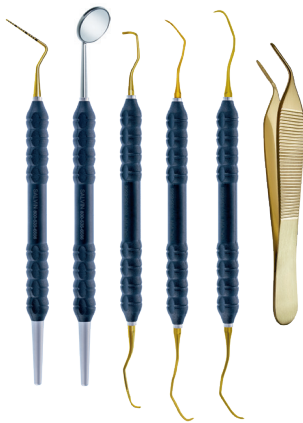

Impla-Safe™ Peri-Implant Management Set

INCLUDES:

- Impla-Safe™ Probe
- Impla-Safe™ Mirror Handle
- Box of 6 Impla-Safe™ Mirrors
- Impla-Safe™ Titanium Curette
- Impla-Safe™ Cement Explorer™

- Impla-Safe™ Cement Remover
- Impla-Safe™ Titanium Micropliers
- Instrument Cassette

#IMPLA-SAFE-INSTR-SET

..... **694.00 SET**

NEW!

Everything For Your Implant Practice But The Implants®

Salvin Dental Specialties, Inc • Toll Free (US & Canada) 800-535-6566 • www.salvin.com

Phone 704-442-5400 • Fax 704-442-5424 • Email orders@salvin.com

© 2023 Salvin Dental Specialties, Inc. All Rights Reserved. REV. 11-2023